



# Getting It Right – Optimum Hygiene and Maintenance



## Preparation of W&H dental transmission instruments and motors

### NOTES ON PREPARATION

These hygiene and maintenance recommendations apply in particular to W&H transmission instruments and motors

Transmission instruments and dental motors labelled with this symbol can be cleaned and disinfected in a thermo washer disinfector.

Transmission instruments and dental motors labelled with this symbol are approved for the sterilization procedure recommended.

#### General notes

- > Please note that transmission instruments must be cleaned and disinfected immediately after every treatment.
- > The use of protective clothing (tear-resistant gloves, goggles, mouth protection) is recommended.
- > Comply with the detailed information in the manufacturer's operating instructions.
- > Use certified and tested disinfectants for W&H transmission instruments and motors and observe the corresponding dosage and reaction time. The disinfectant used must not contain chlorine.
- > Use clean, dry drive air.
- > W&H recommends the use of demineralized water.
- > Please see the detailed information at wh.com.

1

### PREPARATION

1



If necessary, wipe-down disinfection

2




### CLEANING / DRYING



2

#### Manual

#### Machine

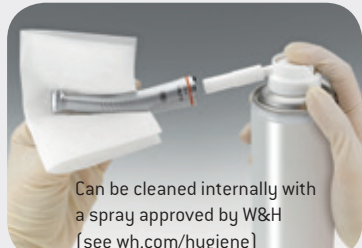

Can be cleaned internally with a spray approved by W&H (see wh.com/hygiene)



3



### DISINFECTION / DRYING



3



wipe-down disinfection

\* The use of a manual process requires a standardized and reproducible cleaning process with confirmed efficacy in every case.



Follow the instructions of the manufacturer of the thermo washer disinfector!



Visual inspection

Visual inspection

4



### LUBRICATION



4

#### Manual

#### Machine



Frequency of lubrication:

- > After every every internal cleaning
- > Before every sterilization
- > After 30 min of use or at least twice daily



electric motors



5



### STERILIZATION

5



W&H recommends sterilization according to EN 13060. Please make sure that the load is dry after both cleaning and sterilization stages!

Pack if necessary

6



### ACCEPTANCE OF LOAD

6



7



### STORAGE

7



Dust-free, clean and dry.

Further information at wh.com/hygiene