

Getting It Right – Optimum Hygiene and Maintenance



**Preparation of W&H dental
 transmission instruments and motors**

Notes on preparation

These hygiene and maintenance recommendations apply in particular to W&H transmission instruments and motors.



Transmission instruments and dental motors labelled with this symbol are approved for the sterilization procedure recommended.



Transmission instruments and dental motors labelled with this symbol can be cleaned and disinfected in a thermo washer disinfector.

General notes

- › Please note that transmission instruments must be cleaned and disinfected immediately after every treatment.
- › The use of protective clothing (tear-resistant gloves, goggles, mouth protection) is recommended.
- › Comply with the detailed information in the manufacturer's operating instructions.
- › Use certified and tested disinfectants for W&H transmission instruments and motors and observe the corresponding dosage and reaction time. The disinfectant used must not contain chlorine.
- › Use clean, dry drive air.
- › W&H recommends the use of demineralized water.
- › Please see the detailed information at wh.com

1

Preparation



2

Cleaning / Drying

Manual



Can be cleaned internally with a spray approved by W&H (see wh.com/hygiene)



Machine



3

Disinfection / Drying



The use of a manual process requires a standardized and reproducible cleaning process with confirmed efficacy in every case.



Follow the instructions of the manufacturer of the thermo washer disinfector!




Visual inspection

Visual inspection

4

Lubrication

Manual

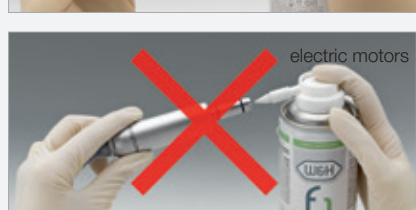

Frequency of lubrication:

- › After every every internal cleaning
- › Before every sterilization
- › After 30 min of use or at least twice daily

Machine



5

Sterilization




Pack if necessary

W&H recommends sterilization according to EN 13060. Please make sure that the load is dry after cleaning and sterilization stages!

6

Acceptance of load

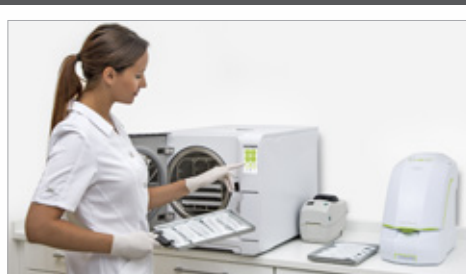

7

Storage



Dust-free, clean and dry.

Further information at wh.com/hygiene