



Quick guide for foot control S-NW


CAN-Dongle



Pairing

The foot control S-NW and the CAN dongle are paired by default.

Connecting CAN dongle

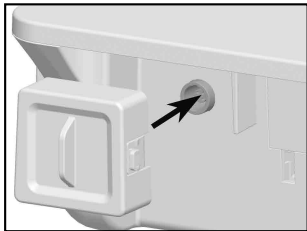


❶ Plug in the CAN dongle.



Pay attention to the positioning!

Removing CAN dongle

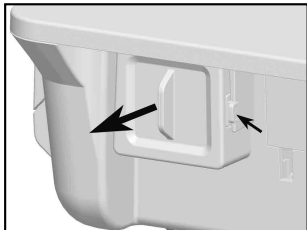

❷ Press the side lock and remove the CAN dongle.

CAN dongle activated



Icon am Display sichtbar

- > CAN dongle inserted
- > Control unit switched on
- > Foot control actuated


SPI-Dongle



Pairing


The foot control S-NW and the CAN dongle are paired by default.

Connecting and disconnecting the SPI dongle

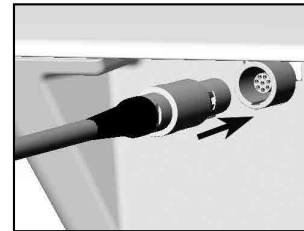

Pay attention to the positioning!

❶ Plug in the SPI dongle or disconnect the SPI dongle from the control unit.

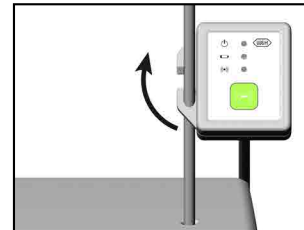

❷ Attach the SPI dongle to the irrigant support or remove the SPI dongle from the irrigant support.



GREEN – SPI dongle activated

LED on if the SPI dongle is connected and the control unit is switched on.

ORANGE – battery

LED flashes if the batteries on the foot control need to be replaced.


BLUE – pairing





Quick guide for foot control S-NW


Changing the batteries

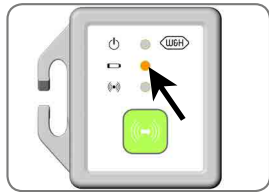


SI-923, SI-915, SA-310, SA-320




SI-1023, SI-1015

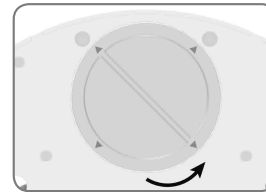

Open the battery compartment




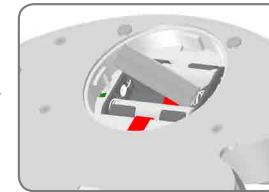


Insert the batteries

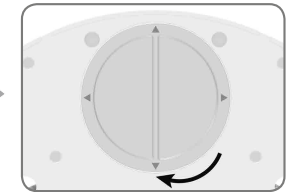


Lock the battery compartment

CAN Dongle Pairing

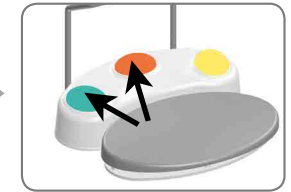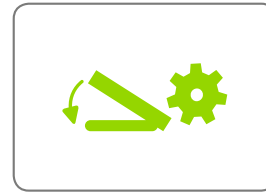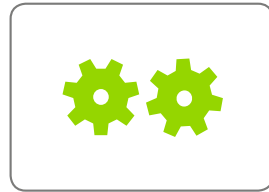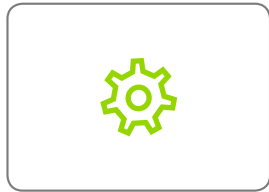


4 seconds


SPI Dongle Pairing

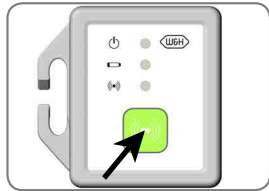


At least 3 seconds

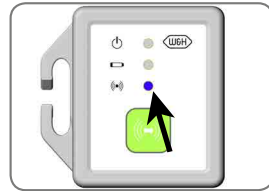

Blue LED flashes

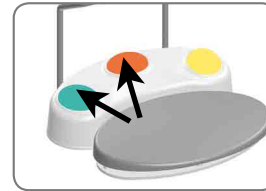


At least 3 seconds

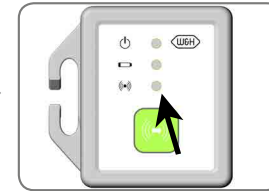

Blue LED off


Switching between control units

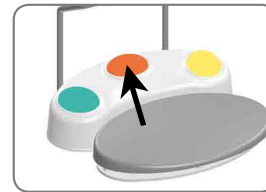

At least 3 seconds



Disable pairing

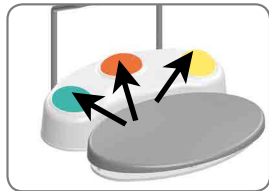

At least 3 seconds